


Using Your Topi-CLICK® Topical Dose Applicator

Step 1: Read Prescription Label

Before using your Topi-CLICK, be sure to read the prescription label for specific dosage instructions, including when and where to apply the medication.

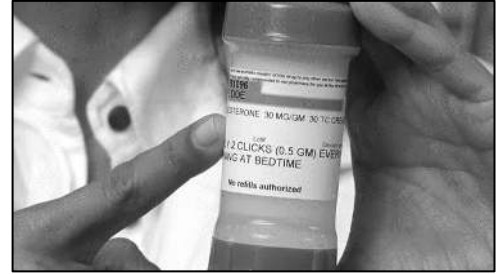

Step 2: Prime Your Topi-CLICK (first use only)

To prime your Topi-CLICK, remove the top. Turn the base until there is a 'click' sound and medication appears on the applicator. Wipe excess medication off with a tissue and dispose of immediately.



Step 3: Turn Base to Activate Medication

Before applying the medication, turn the base the number of clicks noted on the prescription label. This activates the proper dosage amount of the medication.



Step 4: Apply Medication Using the Applicator

Apply the medication **directly to the skin** using the applicator top. This avoids cross-contamination and prevents the medication from transferring to your hands.



Additional Tips and Precautions

1. Allow the application site to dry completely before dressing to avoid the medication transferring to the fabric.
2. Do not share towels with other household members.
3. Alternate the application site to improve the absorption of the medication.



Watch our demonstration video at <https://youtu.be/qhf7KACqjTo> or click on the QR code above.

If you have any questions about the proper use of your Topi-CLICK applicator, please call WeCare Pharmacy at (540) 428-7002.

