

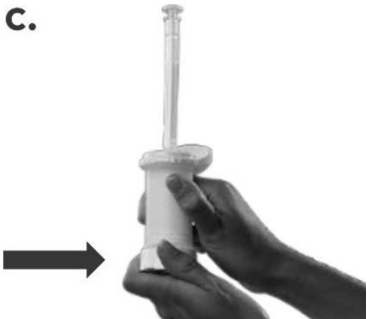



# Using Your Topi-CLICK® PERL™ Vaginal Applicator

<p><b>Step 1: Opening/Attaching the Vaginal Applicator to the PERL.</b></p> <ol style="list-style-type: none"><li>1. Open the PERL shell cap by using your thumb to flip the integrated tab (<b>Figure a</b>).</li><li>2. Snap the applicator to the receptor port on the top of the PERL shell (<b>Figure b</b>).</li></ol> <p><b>NOTE: Before proceeding to the next step, make sure the applicator plunger is fully depressed into the applicator barrel as shown in Figure b and c.</b></p>	 <p>a.</p>  <p>b.</p>
<p><b>Step 2: Loading the Vaginal Applicator.</b></p> <ol style="list-style-type: none"><li>1. To transfer the medication from the PERL to the vaginal applicator, turn ("click") the base of the PERL the number of clicks noted on your prescription label. Each click transfers 0.25mL of cream from the PERL to the vaginal applicator.</li></ol> <p><b>NOTE: It is important to keep the PERL in a <u>vertical position</u> when loading the applicator (Figure c).</b></p> <p><b>Wait <u>15-30 seconds</u> for full dose to load into the vaginal applicator.</b></p>	 <p>c.</p>
<p><b>Step 3: Removing the Vaginal Applicator.</b></p> <ol style="list-style-type: none"><li>1. Slowly bend the applicator as shown in (<b>Figure d</b>) until the applicator releases from the port on the PERL shell.</li></ol> <p><b>NOTE: A small dollop of cream may remain on the top of the PERL port. This is part of your dose.</b></p>	 <p>d.</p>
<p><b>Step 4: Insert Medication Using the Vaginal Applicator.</b></p> <p>Insert the applicator deep into the vagina, and slowly depress the plunger until all medication is inserted. Handwash applicator between doses with warm, soapy water. Not dishwasher safe.</p>	

***If you have any questions about the proper use of your Topi-CLICK PERL vaginal applicator, please call WeCare Pharmacy at (540) 428-7002.***